lionelo
care

User manual
Instrukcja obsługi
Bedienungsanleitung
Инструкция по эксплуатации
Manuale d'uso
Manuel de l'Utilisateur
Manual de usuario
Handleiding
Naudojimo instrukcija
Návod k obsluze

Használati utasítás
Manual utilizare
Bruksanvisning
Bruksanvisning
Brugsvejledning
Käyttöopas
Manual de instruções
Οδηγίες χρήσης
Kasutusjuhend
Lietošanas instrukcija

Fidi Free Lumi

Breast pump

Laktator

Muttermilchpumpe

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

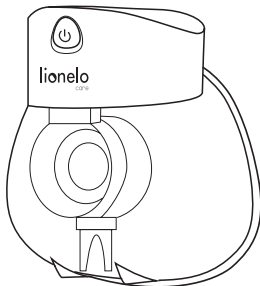
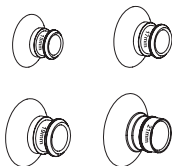
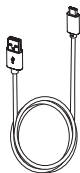
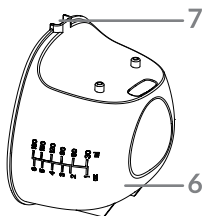
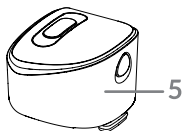
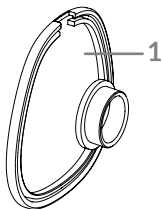
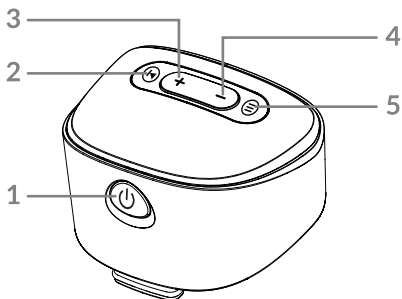
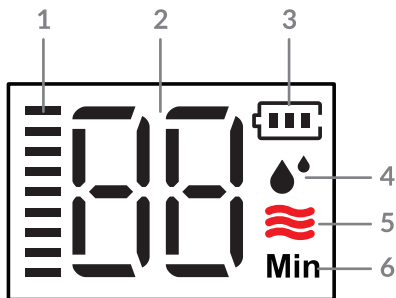
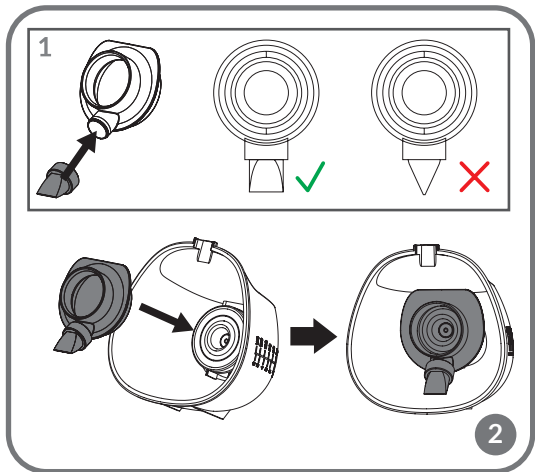
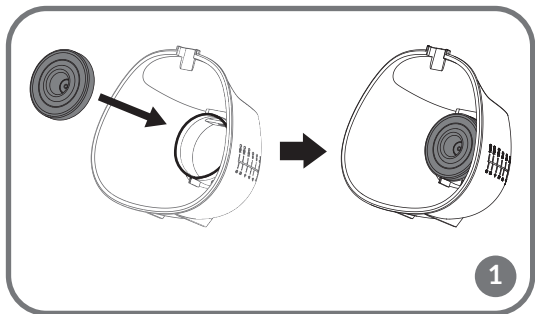
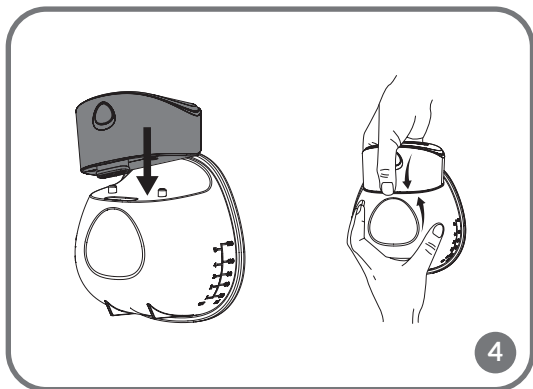
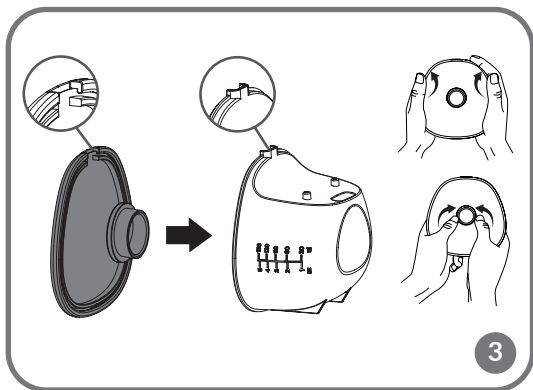
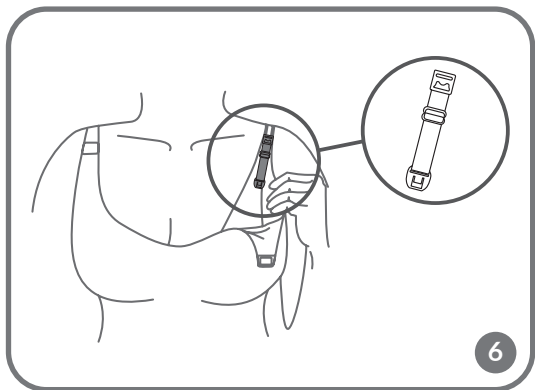
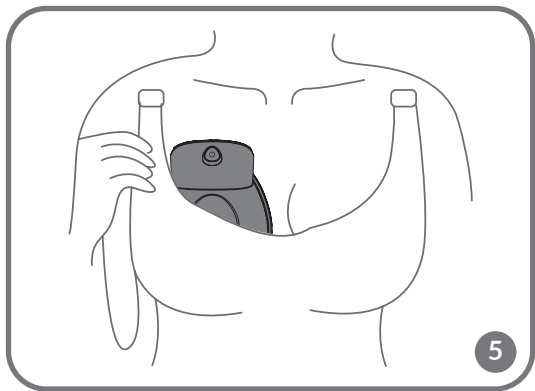
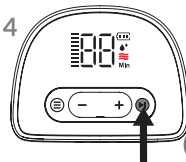
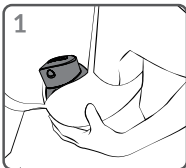
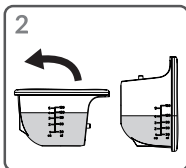
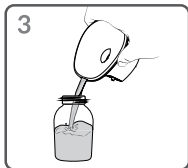
8

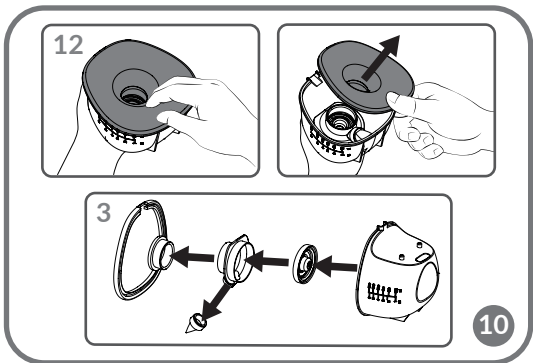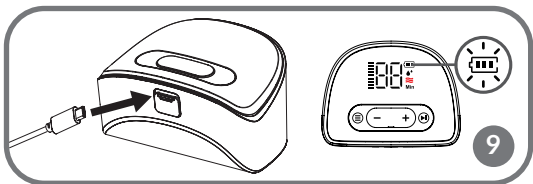Dear Customer!

If you have any questions or comments about the product you purchased, please feel free to contact us by email at: **help@lionelo.com**

Manufacturer:

BrandLine Group Sp. z o.o.

A. Kręglewskiego 1, 61-248 Poznań, Poland

IMPORTANT INFORMATION

Please read these instructions before operating the product to familiarise yourself with its functions and use it as intended.

Failure to observe safety instructions and use contrary to instructions may result in damage to your health.

Keep these operating instructions for future reference.

1. This device may be used by children aged 8 or older and persons with reduced physical, sensory or mental abilities, as well as by persons without relevant knowledge and experience in the use of the device if they are supervised or have been previously instructed in the safe use of the device and the risks involved.
2. This device is not a toy. Do not allow children to play with it. Keep the device out of the reach of children. Cleaning and maintenance must not be performed by children without adult supervision.
3. Do not repair or modify the device. These actions may only be undertaken by an authorised service centre. Use only original spare parts.
4. Do not operate the device if you have noticed any damage. This applies to both the device itself and its accessories, e.g. the USB cable.
5. Do not use the device if it begins to malfunction, has been dropped or submerged in water, becomes excessively hot, or if discolouration, bulging or other abnormal phenomena occur. Contact the manufacturer's authorised service centre.
6. Never leave the device connected to a power socket unattended.

7. The device is not resistant to high temperatures. Use the device away from heat sources, high temperatures, hot surfaces, direct sunlight, sources of spark, open flames, oils and sharp edges.
8. Do not use chemicals to clean the device. Do not wash the main motor unit with water.
9. Switch off the device after each use.
10. Do not immerse the entire device in water or other liquids. Do not use near water.
11. Do not misuse the device.
12. Never handle the device or touch the plug with wet or damp hands. Make sure all breast pump parts are dry before assembly.
13. Before cleaning or moving the device, switch it off and disconnect it from the power source.
14. Before and after each use, wash and disinfect all parts that will come into contact with the body and breast milk.
15. Thoroughly dry each component before use and storage.
16. For hygienic reasons, the device should be used by one person.
17. If using the breast pump causes discomfort or pain, stop using it immediately.
18. Do not use the device throughout your pregnancy.
19. Do not use the device when tired or while sleeping.
20. Special care is required when using the device near children.
21. Do not use the device while lying down or during strenuous exercise or activities that may cause injury.
22. If breast milk does not appear for more than 5 minutes while using the device, stop using the breast pump and consult your doctor.
23. If you have an implanted pacemaker or other electrical device, it is recommended that you consult your doctor before using the device.
24. If the device is used at the maximum permissible ambient temperature of 30°C for 30 minutes, the silicone insert may reach a temperature of 45°C.
25. It is recommended to replace the components that come into contact with breast milk approximately every 3 to 6 months,

depending on usage frequency, and whenever the device has not been used for an extended period.

26. Do not expose the device and its batteries to extreme temperatures. The permissible temperature range in which the battery can be charged is 5°C to 35°C.
27. When the battery is fully charged, immediately unplug the device from the power socket.
28. Do not use breast milk from bottles or containers that have signs of damage on them.
29. The batteries in this device are not replaceable.
30. Be cautious of the risk of short-circuiting the USB-C socket on the pump or battery pack caused by metal objects or liquids.
31. If an electrical product comes into contact with water or other liquids, avoid touching it. If possible, unplug it from the socket, switch it off, and contact the manufacturer.
32. Do not use the device while driving.
33. This device is intended for indoor use only.

Diagram of the device (Fig. A)

- | | |
|---------------------------------------|---|
| 1. Silicone funnel with nipple tunnel | 7. Milk outlet opening |
| 2. Connector | 8. USB cable |
| 3. Silicone membrane | 9. Bra clasp |
| 4. Suction valve | 10. Funnels with diameters of 15 mm, 17 mm, 19 mm, 21 mm, and 24 mm |
| 5. Main body | |
| 6. Milk container | |

Diagram of the main body (Fig. B)

- | | |
|-----------------------------------|--------------------------------------|
| 1. On/off switch | 4. Reducing the intensity level |
| 2. Pause/start button | 5. Switching between operating modes |
| 3. Increasing the intensity level | |

Diagram of the display (Fig. C)

- | | |
|-----------------------------------|------------------------|
| 1. Mode intensity level indicator | 4. Breast pumping mode |
| 2. Operating time indicator | 5. Massage mode |
| 3. Battery level indicator | 6. Operating time unit |

Breast pump modes

1. Massage mode – 9 levels. Massage simulates the initial sucking phase of the baby during breastfeeding.
2. Breast pumping mode – 9 levels. This mode mimics the deeper sucking phase that occurs during breastfeeding.

Assembly of the breast pump

NOTE

Ensure that all parts of the device that come into contact with breast milk are thoroughly washed and disinfected. To prevent damage to the device, ensure that all parts are completely dry.

Wash your hands before assembling the device. Avoid direct contact with the inner surfaces and parts of the device that come into contact with breast milk or the body.

1. Insert the silicone membrane into the milk container (Fig. 1).
2. Attach the suction valve (4, Fig. A) to the connector (2, Fig. A). Ensure the suction valve is fully inserted and correctly oriented (1, Fig. 2).
3. Attach the connector to the silicone membrane in the milk container (Fig. 2). Make sure the connector is pressed down securely.
4. Insert the silicone funnel (1, Fig. A) into the milk container. Attach the silicone funnel to the milk outlet opening (Fig. 3) on the milk container. With both hands, press the nipple tunnel (1, Fig. A) firmly against the connector and align the edges of the funnel collar with the edges of the milk container (6, Fig. A) to create a complete seal. Make sure the funnel collar and nipple tunnel are securely attached.
5. Connect the main body to the milk container (Fig. 4). Ensure that the body is fitted correctly.
6. Firmly press the breast pump against the breast. Make sure the selected funnel fits properly, with the nipple centred within the nipple tunnel (1, Fig. A). Position the breast pump in the bra (Fig. 5).

7. For stiff or inflexible bras, use the included bra clasp. To do this, unhook the bra strap, then attach the clasp and adjust the length according to your preference (Fig. 6).

Milk expression should not be painful. If the breast pump causes discomfort or pain, stop using it immediately.

Ensure that the device fits snugly against the breast, with no gaps between the breast and the silicone funnel. An improper fit will affect the volume of breast milk expressed.

The device can be used with a standard nursing bra and can be worn while sitting or standing. Ensure the device remains securely in place throughout its operation. Otherwise, the operation of the device may be disrupted. Do not lean while using the device.

Directions for use

NOTE

Charge the device for 2 hours before first use.

Wash your hands before use. Ensure that all parts of the breast pump are assembled according to the instructions and that the device is properly fitted to your body.

Avoid using an intensity level that causes discomfort or pain.

1. To turn on the device, press and hold the on/off switch (1, Fig. 7). The device will power on in the last mode used during the previous session.
2. To change operating modes when the device is in use, press the mode switch button (2, Fig. 7).
3. The current operating level of the device is indicated by the mode intensity level indicator (1, Fig. C). Use the decrease and increase intensity buttons (3, Fig. 7) to adjust the mode intensity level between 1 and 9.
4. Press the pause/start button (2, Fig. B) to stop the device (pause mode). Press the button again to resume operation (4, Fig. 7). If the operation is paused for an extended period, the device will automatically turn off after 3 minutes.
5. To activate the backlight and check the amount of breast milk collected, press and hold the pause/start button (4, Fig. 7). Press and hold the button again to turn off the backlight.
6. To turn off the device, briefly press the on/off button (1, Fig. B).

The device will turn off automatically after 30 minutes of operation and lack of activity.

The device has a memory function. When an operating mode is turned off or changed, the device automatically retains the intensity level from the previous use. The time indicator (2, Fig. C) shows the duration of the current use of the device. When the device is in pause mode, the current use time will remain displayed until operation resumes or the device is turned off. To begin a new breast pumping session, turn the device off and then back on by pressing the on/off switch. To find the suitable intensity level for a specific mode, gradually increase it until you feel mild discomfort (not pain!), then reduce the level by one.

Milk transfer

1. While the device is operating, lean forward gently to ensure that all milk from the nipple tunnel drains into the container (1, Fig. 8). Turn off the device.
2. Gently remove the device from the bra and disconnect the main body (2, Fig. 8).
3. Position the milk container so that the rounded part is facing downwards. Next, use the milk outlet opening (7, Fig. A) to pour the milk into the bottle or container designated for storage (3, Fig. 8).

NOTE

Do not use the device to store breast milk. Do not store the device in the refrigerator.

Storing and heating milk

Storage

1. At room temperature (16–25°C), breast milk can be stored for up to 4 hours, or up to 6 hours under exceptionally clean collection conditions.
2. In the refrigerator (4°C), breast milk can be stored for up to 3 days, or up to 5 days under exceptionally clean collection conditions.
3. In the freezer (-18°C), breast milk can be stored for up to 6 months, or up to 9 months under exceptionally clean collection conditions.
4. Do not store breast milk on the refrigerator door.

Defrosting and reheating

1. Freeze milk in food-grade plastic bottles or milk storage bags.
2. To freeze milk, fill the container $\frac{3}{4}$ full, as milk expands in volume when frozen.
3. To preserve the nutritional value of breast milk, defrost it naturally in the fridge (up to 24 hours) or at room temperature (up to 2 hours). You can also defrost breast milk under warm running water at a maximum temperature of 37°C. Breast milk should then be used immediately after defrosting.
4. Before feeding the baby, shake the container with the milk gently to mix it evenly.

NOTE

Always check the temperature of breast milk before feeding.

Do not defrost frozen milk in the microwave or boiling water to avoid nutritional loss and scalding your baby.

Never re-freeze milk that has already been defrosted and store it either at room temperature or in the refrigerator.

Do not use breast milk stored in packages that show signs of damage.

Charging

1. Connect the USB-C cable plug to the power socket on the side of the main body of the device (Fig. 9), and plug the other end with the USB connector into a USB adapter (not included).
1. During charging, the battery level indicator flashes (3, Fig. C). When the flashing stops, it indicates that the device is fully charged.

NOTE

Charge the device for 2 hours before first use.

To ensure the battery's longevity, store the device in a cool location, charge it before the battery fully depletes, and make sure to fully charge it prior to storage.

Charge the device using a 5V DC 1A adapter.

Dismantling the device

1. After detaching the main body, open the milk container from the bottom by gently pulling on the silicone funnel (1, Fig. 10).

2. Detach the silicone funnel from the device by pulling it away, then remove the assembled connector (2, Fig. 10) from the container.
3. Detach the suction valve from the breast pump tee (3, Fig. 10).

Cleaning and disinfection

NOTE

Do not clean or disinfect the main body or the power cord of the device. These parts do not come into contact with food.

Immediately after use, wash all parts that come into contact with the body and breast milk separately to prevent milk residues from drying out and to prevent bacterial growth. Do not wash any parts of the device in the dishwasher.

Before first use

Before using the device for the first time, wash with detergent and then sterilise with boiling water all its parts that will come into contact with breast milk (milk container, tee, valve, and funnels), except the main body. Follow the cleaning and disinfection instructions below.

After each use

1. After use, disassemble the main body and wipe it clean with a soft, damp cloth (1, Fig. 11).
2. After each use, disassemble the device into its individual parts and rinse with warm water all separated parts in contact with the body and with breast milk to remove food residues.
3. Wash all separated parts in warm water with a mild detergent and then rinse thoroughly under clean running water.
4. To disinfect, place all parts in boiling water for at least 10 minutes. Make sure that all parts are immersed in the water and none is directly touching the bottom of the pot.
5. Leave all device parts to dry on a clean towel or paper towel.
6. Store the device parts in a clean bag, container, or other clean location until the next use. If the parts are still wet, do not store them in a sealed container.

NOTE

Do not wash the main body under running water, and do not use harsh cleaners or abrasive materials for cleaning. This could lead to a malfunction of the display. Clean parts that have been in contact with the body once a day and after each use. Be careful not to damage the device when cleaning it.

Do not use sharp tools to clean the silicone parts, as this may result in damage. Be careful not to damage the suction valve during cleaning. If it is damaged, the device will not function properly.

Remove the suction valve from the device carefully. Rinse it with clean water during the cleaning process. Do not insert your finger or any objects into it, as this may cause damage.

Do not use disinfectants to clean the device.

If, after sterilisation in water, you notice a white deposit on parts of the device, this may indicate that the water used is hard. Wipe off the residue using a clean towel and leave the parts to dry on their own.

Troubleshooting

Problem	Solution
The device does not turn on.	• The battery may have depleted or entered the protective mode due to extended storage. Connect the device to the power adapter to activate the battery. • Charge the battery fully before the first use of the device and after prolonged storage.
The running time is not counted down while the pause/start button is pressed.	• Contact an authorised service centre.
The breast pump stopped expressing milk (main body not functioning) after pressing the pause/start button.	• Ensure the device is turned on. • Contact an authorised service centre.

Problem	Solution
Insufficient suction or no suction	• Ensure that all parts of the device are clean, dry, and correctly and securely assembled. Pay particular attention to the correct orientation when installing the suction valve. • Ensure the connector is securely positioned in the container. • Ensure that the silicone funnel of the device fits securely against the breast. • If the suction power does not improve, contact an authorised service centre. • Inspect the device for any cracks or cavities in its components.
Pain or discomfort	• Adjust the intensity level or change the operating mode. • Adjust the position or alignment of the breasts. • If you experience persistent pain or discomfort, consult your doctor.

The photos are for reference only; the actual appearance of products may differ from the photos.

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EN: The product complies with the requirements of European Union directives.

In accordance with Directive 2012/19/EU, this product is subject to separate collection. The product should not be disposed of with municipal waste as it may pose

a risk to the environment and human health. Return the used product to the recycling centre for electrical and electronic equipment.

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